

GIARDIA ANTIGEN TEST KIT

This kit employs lateral immunochromatographic assay in detecting *Giardia lamblia* antigen (*G. lamblia* Ag).

MATERIALS PROVIDED

- ◆ 10 aluminum foil pouches each with a test card and a dropper
- ◆ 10 sample tubes with diluent (buffer)
- ◆ 10 swabs for sample collection
- ◆ literature

STORAGE STABILITY

Test kits stored at 2-30 °C can be used for 18 months after date of manufacture. DO NOT KEEP IN A REFRIGERATOR.


SPECIES:

- ◆ dog
- ◆ cat

SPECIMEN: feces


TEST PROCEDURE

1. Specimen and test kit components should be at room temperature prior to testing.
2. Collect feces with swab & put swab in a sampling tube containing buffer.
3. Swirl the buffer with the swab.

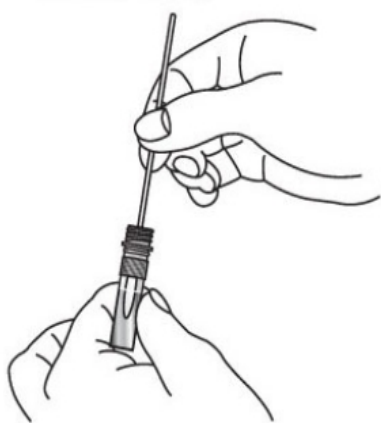

4. Use dropper as pipette to collect the fluid in the sampling tube.
5. Place the test device horizontally.
6. Dispense 4 drops (100 uL) of fluid into the specimen well ("S").

7. Interpret test results within 10 minutes. Results after 10 minutes are invalid.

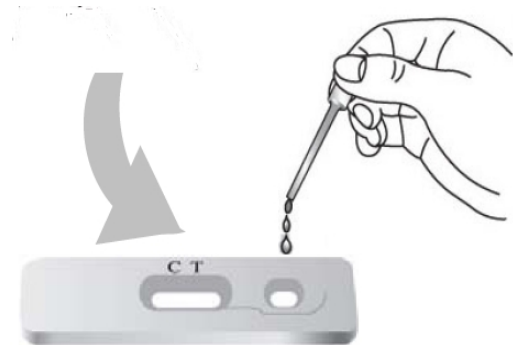

INTERPRETATION OF RESULTS

- ◆ The presence of only 1 color band ("C") in the result window is a **NEGATIVE** result.
- ◆ The presence of 2 color bands ("T" and "C") in the result window is a **POSITIVE** result. It will not matter which color band appeared first. The color bands may be dark or faint in color.
- ◆ If no color band is present, or only the "T" color band is present, the result is considered **INVALID**.



STATISTICS

- ◆ Specificity >99%
- ◆ Sensitivity >99%

DISTRIBUTOR

Aycardo Veterinary Center Inc.
188 N. Bacalso Avenue, Cebu City 6000 Philippines
Telefax: (032) 238-3972
Email: sales@aycardovetcenter.com

MANUFACTURER

VetAll Laboratories, Korea
ISO 9001 Certified